

## Instructions for Use



(1) Fold down outer covering to expose clip and plastic bladder. Push clip in to open, then apply pressure to the sides of the bladder mouth, so that it forms an "O".

(2) Fill with ice to desired level.



(3) Purge Air



(4) Fold mouth of bladder over clip and fasten clip securely. Tuck the clip inside the outer covering and secure the velcro closure.



(5) Apply to patient.

Premium

## COLD PACKS

Item #	Size	Color	Packaging
16100	Large	White	25/box, 4 boxes/case
16101	Small	White	25/box, 4 boxes/case

**Precept**<sup>®</sup>

The Most Comfortable Way to be Safe™

# Premium COLD PACKS



**Precept**<sup>®</sup>

The Most Comfortable Way to be Safe™

## Designed to Conform

Precept's Premium Cold Packs fit around the various joints in the body without restricting movement. This provides maximum coverage without sacrificing comfort. Strong ties provide secure donning of the cold pack. The ties also make it easy to place the cold pack on practically any part of the body.

## Constructed for Convenience

Precept's cold packs come in both small and large sizes for maximum convenience and ease of use.

## Made with Comfort in Mind

Our cold packs are made of 3 layers: a soft outer layer, an absorbent middle layer, and an impervious plastic bladder to hold the ice. The bladder clip is secured within the cold pack and surrounded by the outer 2 layers. This helps prevent the possibility of the clip jabbing or irritating the skin. The absorbent layer keeps the patient dry as the ice melts.

## Making it Easier

Our cold packs feature a conformable opening to allow for easy filling. The opening holds its shape while you pour in the ice, decreasing the chance for spills. The wide mouth also helps make the bag easy to fill.



In some competitive cold packs, the clip is attached to the bag by a single piece of tape that can weaken and even tear, allowing the clip to become detached from the bag. If the clip becomes detached, it can render the ice bag unusable. In Precept's cold packs, the clip is securely attached to the bag by a plastic sleeve or "clip-slip" that is actually an extension of the inner plastic bladder. This "clip-slip" keeps the clip securely in place, thereby preventing it from detaching from the bag.



## Beyond the Average Benefits



The cold pack clip provides a secure seal and prevents accidental leakage. The inner plastic bladder is attached to the soft outer covering with strong, durable stitching. This prevents the bladder from slipping down inside the bag and losing its shape. This feature allows for an even distribution of the cold therapy.

## Providing Superior Cold Therapy

Compared to a leading competitor, Precept's Premium Cold Packs are stronger and more durable, maintaining the cold therapy temperature for a period of almost 40% longer, without needing to be refilled. Our cold packs are also less susceptible to bursting and leakage.

