


Precept N95 Particulate Respirator

Donning Your Respirator

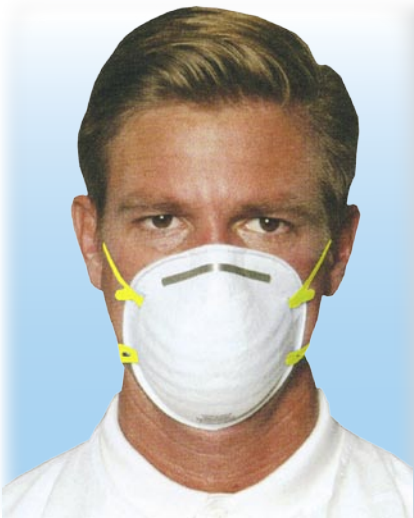

1. Separate the two headbands.
2. Hold the headbands while you cup the respirator to your face.
3. Pull the headbands over your head.
4. Position the lower headband at the base of your neck; the higher headband should go around the crown of your head.
5. Press firmly on both sides of the nosepiece to create a secure, comfortable fit.
6. If necessary, make minor adjustments until the respirator fit feels secure.

In order to obtain the maximum level of protection from your Precept N95 Respirator, a fit test should be conducted prior to the first use. Fit testing should also be conducted on an annual basis and any time there is a change that might affect fit quality (example: facial plastic surgery, excessive weight gain or loss, etc.)